



## Gentle Support for Comfort, Control, and Confidence

### Vaginal Splinting Device

Women with rectocele or pelvic organ prolapse (POP) often struggle with incomplete evacuation and voiding, leading to straining, discomfort, and frequent trips to the toilet. Traditional splinting methods can be inconvenient, unhygienic, and difficult to perform effectively.

Releve® provides a modern, patient-friendly solution with gentle internal support for easier, more complete bowel movements and voiding.



### Why Patients Love Releve®

- **Reduces Straining and Frequent Toilet Trips**  
Prevents stool and urine from becoming trapped, making bowel movements easier and more complete.
- **Effortless 360° Support**  
The silicone balloon adapts to different patients and prolapse sizes, providing a customized fit without the need for manual pressure.
- **Promotes a Natural, Relaxed Posture**  
Designed for comfort, unlike other splinting devices that force awkward positioning, or require direct pressure on the hernia.
- **Easy to Insert and Remove**  
Deflates to a tampon-sized profile.

"I honestly don't know how I would live without it. I have tried other splinting devices, but they don't work for me. The fact that Releve forms to the shape of my particular body has made it much more effective than anything else."

- Beth, Los Angeles, CA

"It has already helped SO many of my patients with bowel dysfunction and prolapse, and there is nothing better. It is so refreshing to find an alternative to digital vaginal splinting, and something that is super easy and user friendly for my patients to operate."


- Kelsey Wisinski, PT, DPT

Developed by a colorectal surgeon, Releve® is a first-line treatment for patients with low to mid-grade prolapse, offering gentle, adjustable support for improved bowel function.




## How Releve Works:

- The deflated balloon is introduced into the vagina prior to a bowel movement or voiding.
- A squeeze bulb allows patients to inflate the balloon to the appropriate size.
- The inflated balloon provides mild, pressurized support to the weakened vaginal walls to prevent the formation of rectocele and/or cystocele.
- The handle allows patients to hold the balloon in place during a bowel movement or voiding.
- Once complete, the device is deflated by pressing the air release button, then removed and cleaned for next use.
- Releve is latex-free.

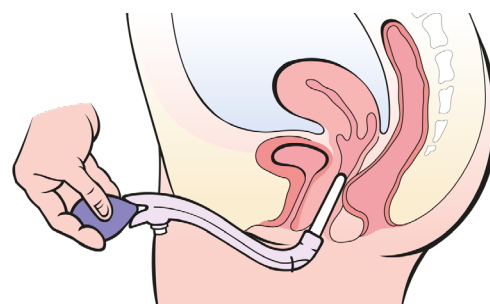

**Deflated Releve Balloon**

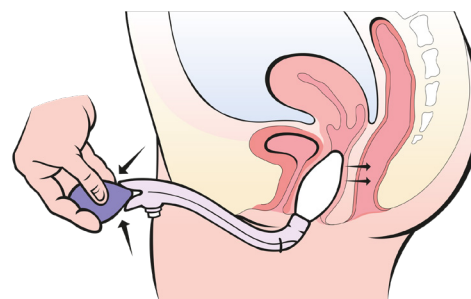

**Inflated Releve Balloon**

## How To Order:

- Order directly from Surgin Inc.  
Tel: (714) 832-6300  
Email: [info@relevsupport.com](mailto:info@relevsupport.com)  
Website: [www.relevsupport.com](http://www.relevsupport.com)
- No prescription required.



Included in the Package:

- 1x Releve Vaginal Splint
- 1x Carrying Bag
- 1x Instructions for Use

